

Shower Stool with Handles

UBAS0022



USER MANUAL



WARNING: Please read this Manual carefully before using the **Shower Stool with Handles** as it contains important information regarding the safe use of the product.

Contents

1.	Description of product	4
2.	Instructions of use	4
	2.1 Adjusting the height	4
3.	Cleaning Instruction	5
4.	Maintainence	5
5.	Product disposal	6
6.	Technical Specifications	6
7.	Warranty	6
8.	Contact Us	7

1. Description of product

The shower stool with handles is intended for use in a bathroom or shower. The shower stool provides a stable seated surface for users with reduced mobility who are unable to stand for a long period of time.



WARNING:

- Ensure these instructions are fully read and implemented. Failure to do so may result in injury. Keep these instructions in a safe place for future reference.
- Maximum user weight of 25st 160kg should not be exceeded.
- Do not attempt to stand or climb on the product.

2. Instructions for use

The product arrives fully assembled. Remove the products packaging and check product carefully.

2.1 Adjusting the height

- (a) Remove the 'e-clip' and slide the leg to the desired position.
- (b) Align the holes and re-insert the 'e-clip'.
- (c) Repeat step (a) and (b) for all four legs.

NOTE:

- If any parts appear damaged do not use and contact your supplier immediately.
- Ensure all legs are set at equal height to avoid instability.

3. Cleaning Instruction

Clean regularly using a general purpose, neutral detergent and warm water.

To disinfect the shower stool, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.



CAUTION:

- Check the suitability of the cleaning products before use.



WARNING:

- Abrasive cleaners or cloths should not be used as these could permanently damage the surface and invalidate any warranty.

4. Maintenance

Check that all 'e-clips' are inserted correctly and that there is no excessive movement.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

5. Product disposal

Product should be thoroughly cleaned prior to disposal.

6. Technical specification

Product Code	Depth	Width	Seat Height	Overall Height
UBAB0022	360 –390mm	495-530mm	470 –625mm	445 – 475mm

7. Warranty

This product is guaranteed to be free of defects in materials or workmanship for one year from date of purchase.

This product has been built to high standards of workmanship and has been carefully inspected prior to shipment. This warranty is an expression of the confidence we have in the workmanship and materials used in our products. In the unlikely event of there being a defect found with the product, please contact your supplier and they will either replace the product or supply parts for the product to be repaired at their discretion.

This warranty does not cover failure of the device caused by misuse, negligence or fair wear and tear of the product.

8. Contact Us

Ultimate Healthcare customer service is available to answer questions from 8.30am to 5.00pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

Ultimate Healthcare Limited

Customer Service, Support, Sales Enquiries and Technical Support
B.B.I.C, Unit 1
Innovation Way
Wilthorpe,
Barnsley,
S75 1JL

Tel: 0333 321 8996

Email: sales@ultimatehealthcare.co.uk

Please Note: Information in this user manual is subject to change without notice and does not represent commitment on the part of Ultimate Healthcare Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Ultimate Healthcare.

Ultimate Healthcare Limited.
Calmore Industrial Estate
Totton, Southampton
SO40 3WW
United Kingdom



Doc no.
UBAS0022.IFU-2

Issue 2.0
December 2022
© Ultimate Healthcare Limited 2022